

HISTAMINE-DRY ICE BLOOD SPECIMEN

Lithium Heparin (green top)

Collection Instructions

COLLECTION REQUIREMENTS

If you are uncertain of the collection procedure after reading these instructions, please contact our Customer Service on 1300 688 522 who will clearly explain the procedure.

IMPORTANT NOTES

- Before proceeding, please read and follow all instructions carefully.
- Please fast for at least 8 hours prior to collection.
- Avoid supplements 24-48 hours prior to collection unless advised by your healthcare practitioner.
- Collect the sample on either a Monday Tuesday or Wednesday.

KIT CONTENTS

Check contents of kit. If items are missing OR you have any questions regarding this kit, please contact Customer Service on 1300 688 522.

- 1 or 2 x 6ml Lithium Heparin Vacutainer (green top)
- 1 x aliquot tube 5ml
- 1 x Specimen transport bag
- 1 x Request form

PO Box 442, Ashburton, VIC, 3147



NutriPATH

BLOOD SPECIMEN (Lith Hep)

COLLECTION KIT



NutriPATH

SPECIMEN COLLECTION

1. NutriPATH has blood collection and dry ice arrangements in place with the following laboratories in each state:

NSW	Douglass Hanly Moir Pathology
VIC	Melbourne Pathology
QLD	QML Pathology
WA	Clinipath Pathology
TAS	Hobart Pathology

PRACTITIONER INFORMATION:

2. Give the patient a NutriPATH Integrative Pathology request form and have them call NutriPATH Integrative Medicine on 1300 688 522 to order a **Lithium Heparin Dry ice TEST KIT** if they have not already been given one.

You must write on the request slip:

'Please on-refer all samples on dry ice to NutriPATH, Melbourne'.

PATIENT INFORMATION STEP 3-5:

3. Ensure that the NutriPATH request form is completely filled out. This includes credit card details, patient information, practitioner details and the test analysis/required.
4. Email NutriPATH the night before of the collection at info@nutripath.com.au stating the collection centre that you will be attending with your name, surname, telephone number and test request.
5. Attend the pathology collection centre as early as possible in the morning (with your blood collection kit) to have the blood specimens

PATHOLOGY COLLECTOR INSTRUCTIONS STEPS 6-12:

6. The pathology collector must **collect the following:**
1 x Lithium Heparin tube
10. The **Lithium Heparin tube** needs to be left as a whole blood sample for testing.
11. Place all specimens in the re-sealable section of the specimen bag and wrap tightly with all paper work included in the pocket of the Biohazard specimen bag.
12. Place all specimens with paperwork in the freezer until dry ice arrives.

PLEASE ONFORWARD THIS TEST TO NutriPATH PATHOLOGY ON DRY ICE

Results will be forwarded to your referring practitioner between 7 to 14 working days, depending on the analyses requested.